



STERIS System 1® Filter Change Log


"A" Filter (Lot #)	"B" Filter (Lot #)	"C" Filter	Air Filter
		Date:	Date:
		Initials:	Initials:
		Date:	Date:
		Initials:	Initials:
		Date:	Date:
		Initials:	Initials:
		Date:	Date:
		Initials:	Initials:
		Date:	Date:
		Initials:	Initials:

Clinical Choice Filter Replacement Instructions

Replacing Filters in STERIS System 1® Pre-Filter Assembly with Clinical Choice Filters

Installation Instructions

- Turn off the water supply by closing the inlet shut-off valve.
- Press the RED Pressure Release Buttons to release residual system pressure.
- Unscrew the housing bowl from the housing body (turn counter clockwise).
- Remove both "A" and "B" cartridges and discard.
- Rinse the housing bowl thoroughly with clean water.
- Installing a new filter:
 - For "A" Filter with Double Open End Two External Gaskets:**
 - Take the Clinical Choice filter out of the box. Hold the filter firmly at the center (along the length of the filter) with the plastic bag still on.
 - Cut open both ends of the bag with a pair of scissors.
 - Wet the gaskets with a small amount of high purity water.
 - Slip the Clinical Choice filter out from the bag into the filter cartridge in the bowl without the fingers touching the filter (as shown with arrow). Discard the bag.
 - For "A" Filter with Single Open End One Internal O-Ring:**
 - Take the Clinical Choice filter out of the box. Hold the filter firmly at the center (along the length of the filter) with the plastic bag still on.
 - Cut open top end (end with O-Rings) of the bag with a pair of scissors.
 - Wet the O-ring with a small amount of high purity water.
 - With the bag still on, seat the Clinical Choice filter into the opening of the cartridge-head and secure it firmly without the fingers touching the filter (direction as show by the arrow). Discard the bag.
 - For "B" Filter:**
 - Take the MedFilters filter out of the box. Hold the filter firmly at the center (along the length of the filter) with the plastic bag still on.
 - Cut open top end (end with O-Rings) of the bag with a pair of scissors.
 - Wet the external O-rings with a small amount of high purity water.
 - With the bag still on, seat the Clinical Choice filter into the opening of the cartridge-head and secure it firmly without the fingers touching the filter (direction as show by the arrow). Discard the bag.
- Replace the filter bowls by screwing the bowl into the filter body (turn clockwise).
- Turn the water back on by opening the shut-off valve.
- Release entrapped air by pressing the RED button on top of the housing bodies.
- All of these steps should be performed by personnel familiar with good laboratory practices.

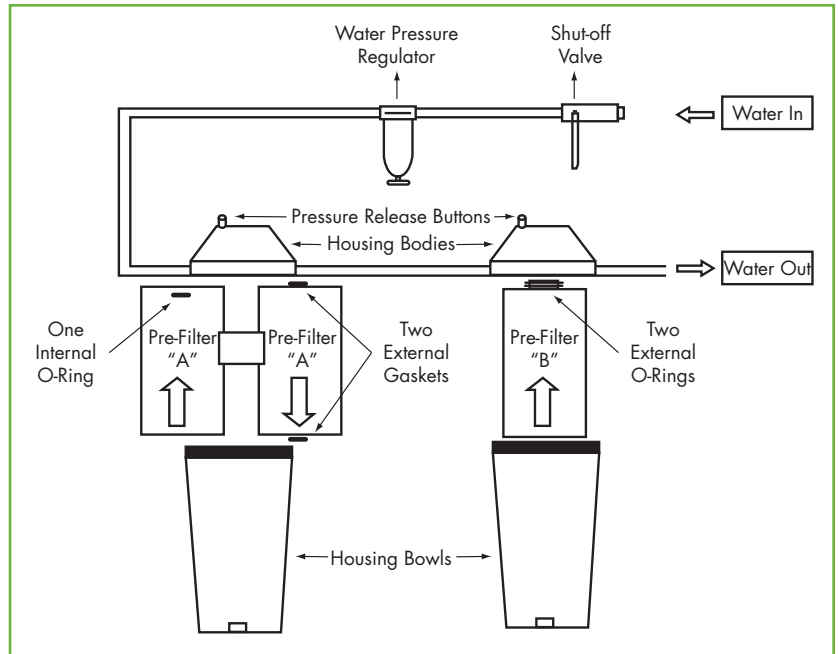

Ordering Information

	"A" Filters		"B" Filter
Description	Single Open End Internal O-Ring	Double Open End Flat Gasket Seal	Single Open End 2-222 O-Ring
STERIS Catalog No.	A1501	—	A1504
Clinical Choice Catalog No.	MFSTEA	MFSTE2A	MFSTEB